



Hillrom™

# IMPROVING MOVEMENT AND INDEPENDENCE

|||||

Your guide to finding the right  
Patient Handling Solutions for  
your paediatric needs.



INTERACTIVE  
e-book

[CLICK HERE TO VIEW FULL SCREEN](#)







Hillrom™

# WELCOME TO THE HILLROM SAFE PATIENT HANDLING EQUIPMENT E-BOOK

We understand you want to specify the best and most appropriate equipment for your younger clients, but the healthcare landscape and paediatric needs are complex and the solutions are many.

To help you navigate which lift, sling, vest or accessory might be the most appropriate to complete the task you and your client require, we have created this E-Book.

It's quick and easy to use, simply click the relevant answer to a set of questions and we will guide you to most appropriate equipment required to complete your task.

For the final stage, simply add your contact details and one of our Hillrom representatives will be in touch to discuss in further detail the perfect solution for your paediatric needs.

[CLICK HERE TO GET STARTED](#)



Hillrom™

# CONTENTS



**9-12**

**Lifting Solutions with multiple carers**

Hillrom HygieneVest™

Hillrom HygieneSling™

Hillrom SilhouetteSling™

Hillrom Original HighBack Sling™

Hillrom Soft Original HighBack Sling™

**14-17**

**Lifting Solutions with a reduced care package**

Hillrom HygieneVest™

Hillrom HygieneSling™

Hillrom SilhouetteSling™

Hillrom Original HighBack Sling™

Hillrom Soft Original HighBack Sling™

**13**

**Standing & stepping transfers with multiple carers**

Hillrom LiftPants™

Hillrom MasterVest™

**18**

**Standing & stepping transfers with a reduced care package**

Hillrom LiftPants™

Hillrom MasterVest™



**What task are you trying to complete with the child?**







**Hillrom™**

**Is there a reduced  
care package  
or multiple  
carers completing  
the task?**



**Reduced care package**



**Multiple carers**







**Hillrom™**

**Does the child  
need to undress  
during this task?**



**Undressing the child**



**Not undressing the child**







**Hillrom™**

**Will the sling be removed after the transfer, or is a breathable fabric required to be left in-situ / for wet environments?**



**Sling will be removed after transfer is complete**



**Sling should be breathable for left in-situ or wet environments**







**Hillrom™**

**Does the child  
need head support  
from the sling  
for this task?**



**Head support needed**



**No head support needed**





Hillrom™

Full seating transfer with multiple carers - undressing & head support required

# Hillrom HygieneVest™ with a Hillrom Golvo™ 9000 Mobile Lift

## OVERVIEW

- For safe lifting and transferring of children to and from the toilet, or when access to the child is required for hygiene purposes and head support is required.

Hillrom mobile lifts have a range of Safe Working Loads and lift heights. The Golvo 9000 mobile lift is recommended for the safe lifting and transfer of children due to its' high lift height, flexible liftstrap and armrests for the client to hold.

## PRODUCT DETAILS

<b>Sizes:</b> XXS, XS, S, M, L, XL	<b>SWL:</b> XXS - L = 200kg, XL = 300kg
------------------------------------	--



[CLICK HERE TO ARRANGE A CLIENT ASSESSMENT](#)







**Hillrom™**

Full seating transfer with multiple carers - undressing & no head support required

## Hillrom HygieneSling™ with a Hillrom Golvo™ 9000 Mobile Lift

### OVERVIEW

- For safe lifting and transferring of children to and from the toilet, or when access to the child is required for hygiene purposes and no head support is needed.

Hillrom mobile lifts have a range of Safe Working Loads and lift heights. The Golvo 9000 mobile lift is recommended for the safe lifting and transfer of children due to its' high lift height, flexible liftstrap and armrests for the client to hold.

### PRODUCT DETAILS

**Sizes:** XXS, XS, S, M, L, XL

**SWL:** XXS - L = 200kg,  
XL = 300kg



[CLICK HERE TO ARRANGE A CLIENT ASSESSMENT](#)





**Hillrom™**

Full seating transfer with multiple carers - head support & breathable fabric required

## Hillrom SilhouetteSling™ with a Hillrom Golvo™ 9000 Mobile Lift

### OVERVIEW

- Provides a reclined sitting posture with comfortable head support. Specifically designed for transfers to narrow wheelchairs where the sling should be left in the chair.
- Thin and pliable without handles or reinforcements.
- Net polyester material makes this sling a feasible option for bathing.

### PRODUCT DETAILS

**Sizes:** XS, S, M, L  
**SWL:** XS - L = 200kg

Hillrom mobile lifts have a range of Safe Working Loads and lift heights. The Golvo™ 9000 mobile lift is recommended for the safe lifting and transferring of children due to its high lift height, flexible liftstrap and armrests for the client to hold.



## Hillrom Original HighBack Sling™ with a Hillrom Golvo™ 9000 Mobile Lift

### OVERVIEW

- Provides a reclined sitting posture with head support, an ideal option for children with poor head and torso stability.
- Suitable for lifting to and from the floor.
- Net polyester material makes this sling a feasible option for bathing.

### PRODUCT DETAILS

**Sizes:** XS, S, MS, M, L, XL, XL  
**SWL:** XS - L = 200kg, XL = 500kg

Hillrom mobile lifts have a range of Safe Working Loads and lift heights. The Golvo™ 9000 mobile lift is recommended for the safe lifting and transferring of children due to its high lift height, flexible liftstrap and armrests for the client to hold.



[CLICK HERE TO ARRANGE A CLIENT ASSESSMENT](#)





**Hillrom™**

Full seating transfer with multiple carers - head support required

## Hillrom Soft Original HighBack Sling™ with a Hillrom Golvo™ 9000 Mobile Lift

### OVERVIEW

- Provides a reclined sitting posture with head support, an ideal option for children with poor head and torso stability.
- Suitable for lifting to and from the floor.

### PRODUCT DETAILS

**Sizes:** XS, S, M, L, XL, XXL

**SWL:** XS - L = 200kg,  
XL = 500kg

**Hillrom** mobile lifts have a range of Safe Working Loads and lift heights. The Golvo™ 9000 mobile lift is recommended for the safe lifting and transferring of children due to its high lift height, flexible liftstrap and armrests for the client to hold.



[CLICK HERE TO ARRANGE A CLIENT ASSESSMENT](#)



**Hillrom™**

Standing and stepping transfers with multiple carers

## Hillrom LiftPants™ with a Hillrom Golvo™ 9000 Mobile Lift

### OVERVIEW

- With Hillrom LiftPants™ Ambulation Support the child can gait train under secure conditions.
- Provides freedom of movement, while relieving some or all of the burden of body weight.
- Supports safely, allowing the child to move on their own without the risk of falling.

### PRODUCT DETAILS

**Sizes:** XS, S, M, L, XL

**SWL:** XS - XL = 200kg

Hillrom mobile lifts have a range of Safe Working Loads and lift heights. The Golvo™ 9000 mobile lift is recommended for the safe lifting and transferring of children due to its high lift height, flexible liftstrap and armrests for the client to hold.



## Hillrom MasterVest™ with a Hillrom Golvo™ 9000 Mobile Lift

### OVERVIEW

- Facilitates sit-to-stand movement of children who are able to support their own weight when standing.
- Ideal aid for standing, gait and balance training.
- When transferring to or from the toilet, the lower body is free for dressing / undressing.
- Option of an additional leg harness to provide additional support through the pelvis.

### PRODUCT DETAILS

**Sizes:** XS, S, MS, M, L, XL, XXL

**SWL:** XS - L = 200kg, XL-XXL = 500kg

Hillrom mobile lifts have a range of Safe Working Loads and lift heights. The Golvo™ 9000 mobile lift is recommended for the safe lifting and transferring of children due to its high lift height, flexible liftstrap and armrests for the client to hold.



[CLICK HERE TO ARRANGE A CLIENT ASSESSMENT](#)





**Hillrom™**

Full seating transfer with a reduced care package - undressing & head support required

## Hillrom HygieneVest™ with a Hillrom Likorall Overhead Lift

### OVERVIEW

- For safe lifting and transferring of children to and from the toilet, or when access to the child is required for hygiene purpose and head support is required
- Likorall overhead lift systems have a range of Safe Working Loads from 200kg (31 stone) to 500kg (80 stone). The lift motors can be supported by installed rails or by a FreeSpan gantry system

### PRODUCT DETAILS

**Sizes:** XXS, XS, S, M, L, XL

**SWL:** XXS - L = 200kg, XL = 300kg



[CLICK HERE TO ARRANGE A CLIENT ASSESSMENT](#)



**Hillrom™**

Full seating transfer with a reduced care package - undressing & no head support required

## Hillrom HygieneSling™ with a Hillrom Likorall Overhead Lift

### OVERVIEW

- For safe lifting and transferring of children to and from the toilet, or when access to the child is required for hygiene purposes and no head support is needed.
- Likorall overhead lift systems have a range of Safe Working Loads, from 200kg (31 stone) to 500kg (80 stone). The lift motors can be supported by installed rails or by a FreeSpan gantry system

### PRODUCT DETAILS

**Sizes:** XS, S, M, L, XL

**SWL:** XXS - L = 200kg, XL = 300kg



[CLICK HERE TO ARRANGE A CLIENT ASSESSMENT](#)





**Hillrom™**

Full seating transfer with a reduced care package - head support & breathable fabric required

## Hillrom SilhouetteSling™ with a Hillrom Likorall Overhead Lift

### OVERVIEW

- Provides a reclined sitting posture with comfortable head support. Specifically designed for transfers to narrow wheelchairs where the sling should be left in the chair.
- Thin and pliable without handles or reinforcements.

### PRODUCT DETAILS

**Sizes:** XS, S, M, L  
**SWL:** XS - L = 200kg

Likorall overhead lift systems have a range of Safe Working Loads, from 200kg (31 stone) to 500kg (80 stone). The lift motors can be supported by installed rails or by a FreeSpan gantry system



## Original HighBack Sling™ with a Hillrom Likorall Overhead Lift

### OVERVIEW

- Provides a reclined sitting posture with head support, an ideal option for children with poor head and torso stability.
- Suitable for lifting to and from the floor.
- Net polyester material makes this sling a feasible option for bathing.

### PRODUCT DETAILS

**Sizes:** XS, S, MS, M, L, XL  
**SWL:** XS - L = 200kg, XL = 500kg

Likorall overhead lift systems have a range of Safe Working Loads, from 200kg (31 stone) to 500kg (80 stone). The lift motors can be supported by installed rails or by a FreeSpan gantry system



[CLICK HERE TO ARRANGE A CLIENT ASSESSMENT](#)



**Hillrom™**

Full seating transfer with a reduced care package - head support required

## Hillrom Soft Original HighBack Sling with a Hillrom Likorall Overhead Lift

### OVERVIEW

- Provides a reclined sitting posture with head support, an ideal option for children with poor head and torso stability.
- Suitable for lifting to and from the floor.

### PRODUCT DETAILS

**Sizes:** XS, S, M, L, XL, XXL

**SWL:** XS - L = 200kg,  
XL = 500kg

Likorall overhead lift systems have a range of Safe Working Loads, from 200kg (31 stone) to 500kg (80 stone). The lift motors can be supported by installed rails or by a FreeSpan gantry system



[CLICK HERE TO ARRANGE A CLIENT ASSESSMENT](#)





**Hillrom™**

Standing and stepping transfers with a reduced care package

## Hillrom LiftPants™ with a Hillrom Likorall Overhead Lift

### OVERVIEW

- With Hillrom LiftPants™ Ambulation Support the child can gait train under secure conditions.
- Provides freedom of movement, while relieving some or all of the burden of body weight.
- Supports safely, allowing the child to move on their own without the risk of falling.

### PRODUCT DETAILS

**Sizes:** XS, S, M, L, XL

**SWL:** XS - XL = 200kg

Likorall overhead lift systems have a range of Safe Working Loads, from 200kg (31 stone) to 500kg (80 stone). The lift motors can be supported by installed rails or by a FreeSpan gantry system



## Hillrom MasterVest™ with a Hillrom Likorall Overhead Lift

### OVERVIEW

- Facilitates sit-to-stand movement of children who are able to support their own weight when standing.
- Ideal aid for standing, gait and balance training.
- When transferring to or from the toilet, the lower body is free for dressing / undressing.
- Option of an additional leg harness to provide additional support through the pelvis.

### PRODUCT DETAILS

**Sizes:** XS, S, MS, M, L, XL, XLL

**SWL:** XS - L = 200kg, XL-XXL = 500kg

Likorall overhead lift systems have a range of Safe Working Loads, from 200kg (31 stone) to 500kg (80 stone). The lift motors can be supported by installed rails or by a FreeSpan gantry system



[CLICK HERE TO ARRANGE A CLIENT ASSESSMENT](#)



**ABOUT HILLROM**

Hillrom is a global medical technology leader whose 10,000 employees have a single purpose: enhancing outcomes for patients and their caregivers by advancing connected care. Around the world, our innovations touch over 7 million patients each day. They help enable earlier diagnosis and treatment, optimize surgical efficiency and accelerate patient recovery while simplifying clinical communication and shifting care closer to home. We make these outcomes possible through connected smart beds, patient lifts, patient assessment and monitoring technologies, caregiver collaboration tools, respiratory care devices, advanced operating room equipment and more, delivering actionable, real-time insights at the point of care. Learn more at [hillrom.com](https://hillrom.com).

For more information, please contact your local distributor or Hillrom sales representative.

**hillrom.com**

Hillrom range of lifts and accessories are intended to be used for the lifting and transferring of patients in a variety of care settings. Hillrom slings and accessories are intended to be used in combination with Hillrom lifts for a range of lifting and transferring situations in a variety of care settings.

Class 1

Manufacturer: Liko AB, Nedre vägen 100, 975 92 Luleå, Sweden

These products are regulated health care products which, where required by applicable regulations, bear a CE mark. Hill-Rom recommends you carefully read the detailed instructions for safe and proper use included in the documentation accompanying medical devices. The personnel of healthcare establishments are responsible for the proper use and maintenance of these medical devices.

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.

© 2021 Hill-Rom Services, Inc. ALL RIGHTS RESERVED. APR223501 MAY-2021 ENG – UK

