

Envision[®] Low Airloss Therapy Surface Quick Tips

CPR Mode

The CPR control is located on front of air supply unit in upper right-hand corner.


- When CPR is pressed, the system hyper-inflates to supply a firm surface for CPR.
- To turn CPR mode on, press the CPR control on air supply unit.
- CRP mode will remain active for 60 minutes:
 - To end CPR mode before 60 minutes, on CPR screen, press “Stop.” A message will show to indicate the mattress has returned to standard pressure redistribution. Press “OK.”



Note: The CPR control does not control the bed. Bed will have to be placed into position for CPR separately using bed controls.

New Patient Setup – Entering Patient Weight

- To enter weight:
 1. Press “**Enter Patient Weight**”.
 2. Select **lb** or **kg**.
 3. Enter weight.
 4. Press “**Enter**” to accept weight.
 5. Press “**Done**” to exit the Enter Patient Weight screen.

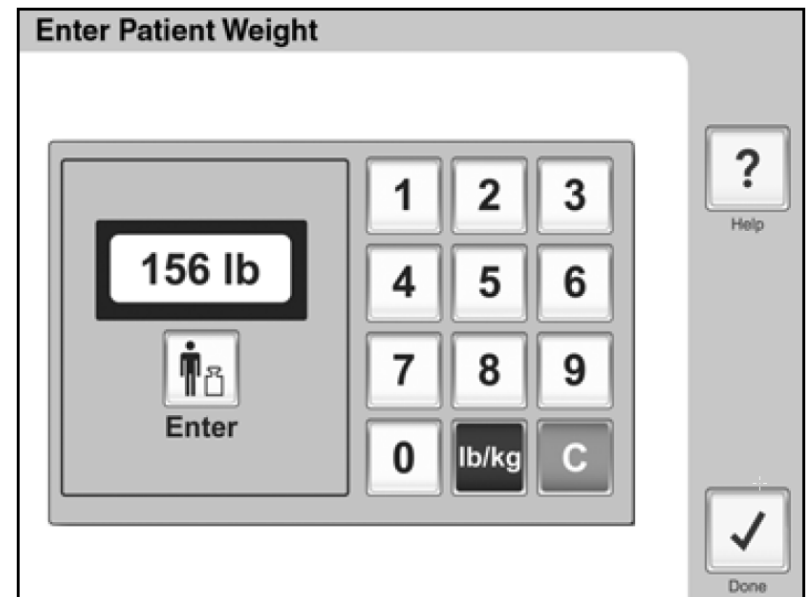

Note: The patient's weight must be entered for correct pressure redistribution.

Max Inflate

To help make it easier to reposition in bed, Max Inflate can be used.

- To turn on:
 - Press the “**Enable**” control.
 - Press “**Max Inflate**” control.
- Mattress will remain max inflated for 30 minutes. After 30 minutes, mattress automatically returns to previous pressure settings.
 - To turn off before 30 minutes, press “Enable” control and then press “Max Inflate” control.



Turn Assist™ Mode

Turn Assist™ Mode helps caregivers temporarily turn patient for bedpan, dressing changes or other procedures.

To engage Turn Assist™ Mode:

- Ensure patient is centered on the mattress.
- Verify siderails are raised and locked and HOB is below 30°.
- Press Enable control then press Turn Assist™ Mode Left or Turn Assist™ Mode Right control.
- When patient and bed are ready for Turn Assist, press “Start”.
- Therapy surface turns patient to selected side, beeps one time, and then holds the patient in turned position.



**Turn Assist™
Mode Right**



**Turn Assist™
Mode Left**

Note: Before activating Turn Assist, make sure siderails are in the up and locked position.


Bed Exit and Alarm

There are two settings for Bed Exit Alarm:

- **Patient Movement** – sounds when patient moves away from head section, such as sitting up in bed.
- **Out of Bed** - sound when the patient's presence is no longer detected on the surface.

To engage alarm: Press and hold applicable alarm control. Once engaged, the icon for alarm will show on the main screen.

If alarm sounds, a message screen will show on the main screen. Follow instructions on screen. After 15 minutes, if alarm is silenced but not reset, the alarm will sound again.

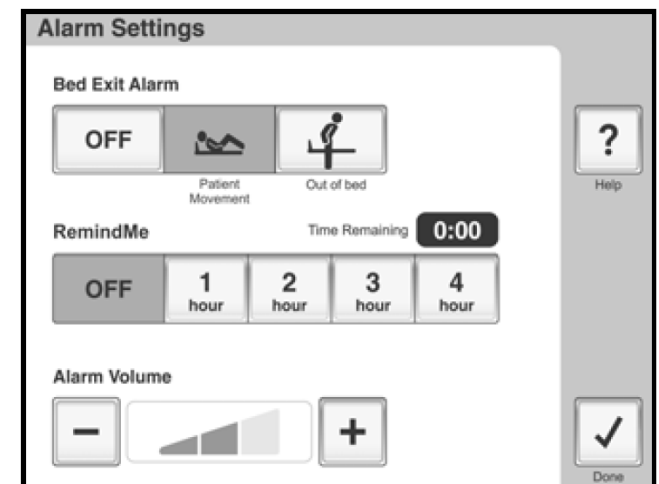

Note: Bed Exit Alarm is a local alarm and not connected to facility's nurse call system.

Quick Study In-service System

Quick Study In-service System supplies in-service data for caregivers to reference.

1. Press Quick Study In-service System.
2. Press the applicable subject.
3. Press the necessary topic on menu.
4. To see an in-service video:

- Press Show Me.
- Press Done to return to the Quick Study In-service System topic screen.

