



The ABC's of Propaq LT

<p>A</p> <p>Age Ranges</p>	<p>Neonate: Term birth (37 weeks) through 28 days OR up to 44 weeks gestational age Pediatric: 29 days to 12 years of age Adult: 13 years of age and older</p>
<p>B</p> <p>Battery Info</p>	<p>Uses a Lithium Ion battery Duration: <ul style="list-style-type: none"> • ECG/Resp only: ~24 hours • ECG/Resp/SpO2/BP every 15 minutes: 8hours Durations are in Standalone AND Acuity connected, continuous monitoring, and minimal display use (defined as 5 minutes display use per hour).</p>
<p>C</p> <p>Cleaning</p>	<p><u>Device</u> Wipe with a nearly-dry cloth moistened with an approved cleaning solution: <ul style="list-style-type: none"> • Warm water, liquid soap, Coverage®, Windex®, Ovation®, Hydrogen peroxide, Wex-cide®, T.B.Q.® After cleaning, remove excess solution with a soft cloth dampened with clean water, then dry. Do not let solution get into connector openings or crevices. Do not autoclave or immerse.</p> <p><u>Accessories</u> Consult manufacturer's instructions.</p>
<p>D</p> <p>Defaults</p>	<p>Create default settings using the PC Configurator Tool.</p>
<p>E</p> <p>End Monitoring</p>	<p><u>Standby/Temporary Disconnect</u></p> <ol style="list-style-type: none"> 1. Remove cables/leads from device. 2. Acknowledge equipment alert by pressing Select . 3. At Acuity, the patient's waveform tile will turn BLUE indicating 'Standby'. 4. To resume monitoring, use good lead prep/placement to apply leads, and then plug cables back into top of device. <p><u>Power down/Permanent Disconnect</u></p> <ol style="list-style-type: none"> 1. Press the power button on the top right hand side of the monitor. 2. When prompted, press  to select "Delete and Shut Down." 3. Remove leads from patient.