Welch Allyn [®]
23860 Diagnostic Veterinary Oscope with Throat Illuminator
Material No. 705240 Rev. C

For guaranteed performance, use only Welch Allyn lamps and specula.

Symbols

Attention. Read Operating Manual for Cautions and Instructions for Use.

TYPE BF – Indicates this is a product with Type BF applied parts.

Do not dispose of this product as unsorted municipal waste. Prepare this product for reuse or separate collection as specified by Directive 2002/96/EC of the European Parliament and the Council of the European Union on Waste Electronic and Electrical Equipment (WEEE). If this product is contaminated, this directive does not apply.

For more specific disposal information, see www.welchallyn.com/weee, or contact Welch Allyn Customer Service at +44 207 365 6780.

Warnings/Cautions

CAUTION: To minimize external otoscope housing temperatures, on-time should not exceed 2 minutes and off-time should not be less than 10 minutes.

CAUTION: Federal Law restricts this device to sale by or on the order of a physician.

WARNING: This instrument is not intended to be used in the eye.

Operating Instructions

Attaching Oscope Head to power source

(Figure 1)

Align the cutouts in the Oscope Head base with lugs on power source. Push in slightly and turn 90°. The Oscope is now ready to use. To remove, reverse this procedure. The Oscope Heads fit all 3.5 v. Welch Allyn power sources.

Using Focus

(Figure 2)

Use the focus in the Default or Free position.

Default Focus: align the green line on the focusing wheel to the green dot on the side of the otoscope.

Free Focus: rotate the focus wheel (A) or (B) to zoom in or out.

Zoom in: rotate focus wheel towards longer dashes.

Zoom out: rotate focus wheel towards shorter dashes.

Attaching Specula

(Figure 3)

To attach the specula onto the otoscope, twist the specula clockwise into the TipGrip (C) until it locks.

Removing Specula

To remove the specula, turn the TipGrip (C) counterclockwise. Or grab the specula and twist it counterclockwise.

Note: Oscope only supports Welch Allyn specula.

Using Throat Illuminator for REF 23860

(Figure 4)

- Grasp the otoscope section with the left hand.
- With the right hand, grasp the throat illuminator’s collar at (D) (under the silver ring).
- While holding the throat illuminator collar stationary, twist the otoscope section counterclockwise at (D) until the otoscope head is removed. The throat illuminator is now ready for use.
- To replace otoscope section, reverse the steps.

Note: It is important to note that while the throat illuminator section provides an excellent source of general illumination, avoid more than momentary direct skin contact due to possible heat build up.

Lamp Replacement

(Figure 5)

CAUTION: Halogen lamps are pressurized to provide maximum efficiency and illumination. Mistreatment may cause shattering. Protect lamp surface against abrasions and scratches. Use protective eyeglasses when handling. Verify power is off when replacing lamp. Dispose of lamp with care.

CAUTION: Lamps may be hot. Allow the lamps to cool before removing.

Cleaning Recommendation

Repairs and cleaning (with the exception of lamp replacement) beyond what is specified below must be performed at the factory. Do not apply alcohol, chemicals, or water to the lenses. Solution entering the optical assembly could damage internal components.

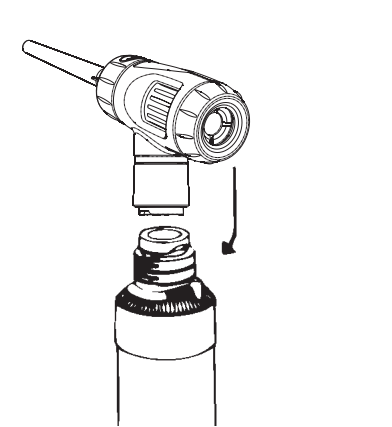
Clean the otoscope head or throat illuminator with a dry cloth.

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