

Welch Allyn Connex[®] Vital Signs Monitor Quick Reference Card




Power on/Standby




- Powers up the monitor
- Brief press – Sets monitor to Display power saving mode

Battery status

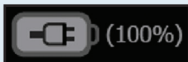

Charging

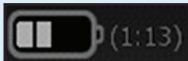


Approximate operating time remaining



Battery removed or not holding a charge

Input clinician ID with barcode scanner

1. Touch the Clinician ID section (left edge) of the Device Status area.

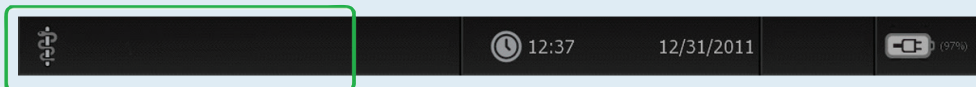

2. Scan the clinician ID with a barcode scanner.
The ID appears in the field.
3. Touch **OK**.


Input patient ID with barcode scanner

1. Touch the **Home** tab.
2. Scan the patient's ID with a barcode scanner.
The ID appears in the field.

WelchAllyn[®]


Advancing Frontline Care[™]

Reset (pause or turn off) audio alarms

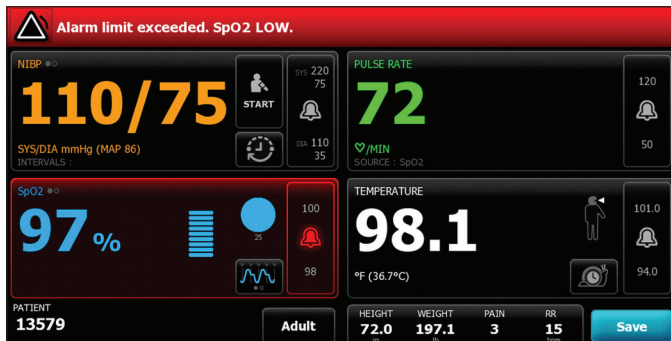

Touch .

Adjust alarm limits

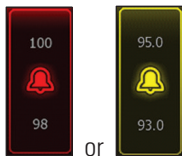

Touch  or  in the alarming parameter to display alarm controls. Adjust a limit or turn off.

Start/Stop blood pressure



Start automatic intervals

Intervals Device Clinician Profiles Advanced


Automatic
 Program
 Stat

Minutes
15
(1 - 240)

Automatic print on interval

Start intervals

Home Patients Alarms Review Settings

1. Properly size the blood pressure cuff and position it around the patient's bare upper arm.
2. On the Home tab, touch .
3. Select **Automatic**.
4. Use the numeric keypad to enter the length of time between NIBP measurements.
5. Touch **Start intervals**.

Enter manual parameters

1. From the Home tab, touch anywhere within the Manual parameters frame. The Manual tab appears.




2. Touch the up/down arrow keys or the keypad to manually adjust height, weight, pain level, temperature, respiration rate, or other parameters.
3. Touch **OK** to confirm settings and return to the Home tab.

Note Entries will not be saved unless you touch **OK**.

Set modifiers

1. Touch the **Patients** tab.

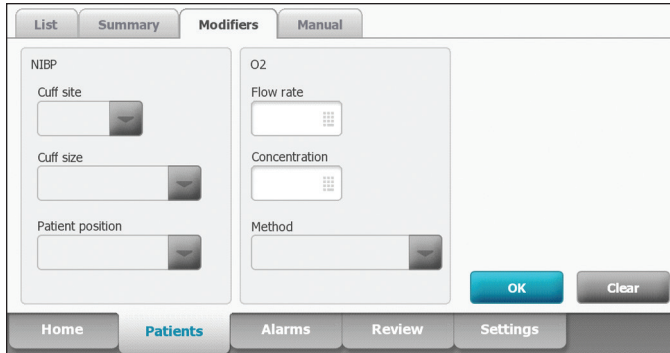

2. Touch the **Modifiers** tab.
3. Adjust the settings as required.
4. Touch **OK** to accept or **Clear** to delete all entries.

Note Entries will not be saved unless you touch **OK**.

Save patient data

After taking patient measurements, touch **Save**.
A message will appear indicating a successful or failed save.

Manage patient records

Patient records can be sent to the network, printed, or deleted.

1. Touch the **Review** tab.

<input checked="" type="checkbox"/>	Patient	Date / Time	NIBP	Temp	PR	SpO2	SpHb	Ht. Wt. P. RR
<input checked="" type="checkbox"/>	13579	12/31/2011 11:48	131/81(98)	97.4	81	96	14.2	72.0/197.1/3/15
<input checked="" type="checkbox"/>	13579	12/31/2011 11:49	130/86(100)	98.3	82	96	14.2	72.0/197.1/3/15
<input checked="" type="checkbox"/>	13579	12/31/2011 11:51	128/88(101)	98.5	81	95	14.1	72.0/197.1/3/15
<input checked="" type="checkbox"/>	13579	12/31/2011 11:52	132/88(103)	98.1	84	96	14.0	72.0/197.1/3/15
<input checked="" type="checkbox"/>	13579	12/31/2011 11:56	132/78(96)	98.4	83	97	14.0	72.0/197.1/3/15

2. Select records by touching the check box.
3. Touch **Send** to transmit the records to the network, **Print** to print the records, or **Delete** to permanently remove the records.

Note The  icon indicates the records have been sent to and received by the network.

Technical Assistance

USA +1 800 535 6663
+1 315 685 4560
Canada +1 800 561 8797
Australia +61 2 9638 3000
China +86 21 6327 9631

European
Call Center +353 46 90 67790
France +33 155 69 58 49
Germany +49 695 098 5132
Italy +39 026 968 2425


Japan +81 42 703 6084
Latin America +1 305 669 9003
Malaysia +603 7875 3341
Netherlands +31 202 061 360
Singapore +65 6419 8100

South Africa +27 11 777 7555
Spain +34 917 499 357
Sweden +46 85 853 6551
United Kingdom +44 207 365 6780

5 Power plug retainer

6 Scanner cord clip

7 Power cord wrap

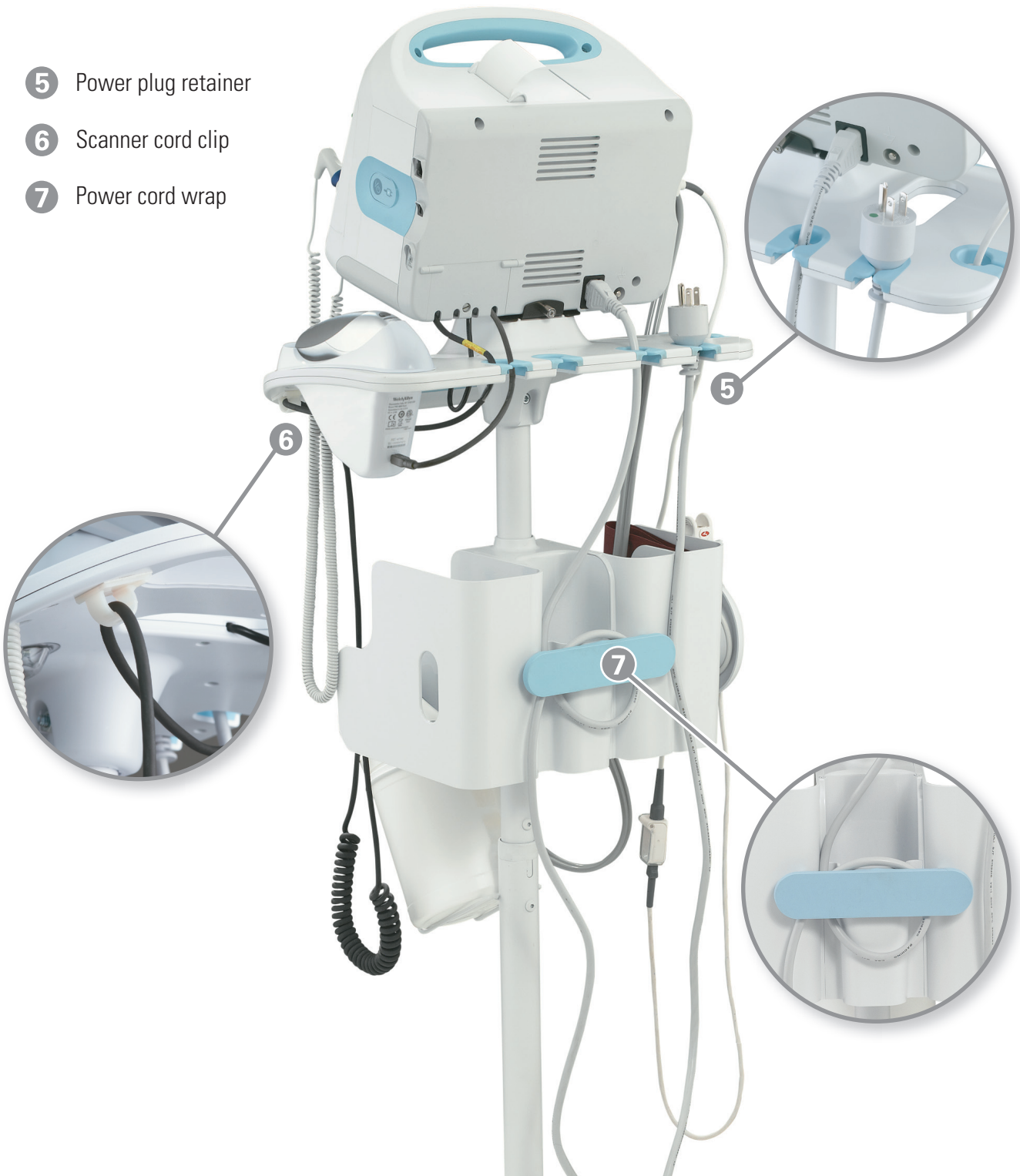

Accessory Cable Management Stand

Quick Reference Card

- 1 LED light button
- 2 SpO2 holder
- 3 Barcode scanner
- 4 Wire bracket for cleaning wipes



REF 103502

Mat. No. 717131, 80016619 Ver. B

www.welchallyn.com