Dear Welch Allyn CP 150™ 12-lead electrocardiograph owner:

Enclosed is a new software update, version 2.00.03, to improve the functionality of your electrocardiograph. The improvements for this update are a revision to the translation of beats per minute (bpm) in Danish, German, Finnish, French, and Dutch and calibration support for a new transducer design.

Updating your CP 150 software to version 2.00.03

Before you begin

To perform the update, you will need the following:

- The USB drive that contains the update utility.
- The CP 150 you plan to update.
- A cable to connect the CP 150 to AC power.
- An AC power outlet.

Update the software

1. From the ECG home tab, touch the **Settings** tab. The ECG tab and the vertical ECG configuration tab appear.
2. Touch the **Device** tab.
3. Touch the **Updates** tab.
4. Insert a USB flash drive containing the latest software into a USB port on the back of the device and touch the **Update** button.

"Update complete. Press OK to reboot." appears after the update is complete.
5. Remove the USB drive from the CP 150.
6. Touch **OK**.

The device automatically reboots.

Note Do NOT remove the USB drive or AC power from the device during reboot.

To verify the software version:

- a. Touch the **Settings** tab.
- b. Touch the **Device** tab.
- c. Touch the **About** Tab.
- d. Verify that the correct software version is listed for the CP 150. You are ready to use the updated software.

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