


**Welch Allyn® RetinaVue® 700 Imager
Battery Charger Assembly Instructions**


Mat. No. 728695, 80024084 Ver. C
Revision date: 2019-04

Welch Allyn Technical Support: www.welchallyn.com/support

107144

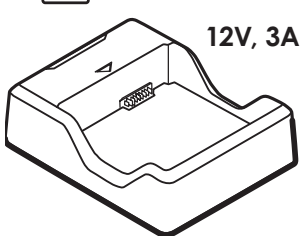


12V, 3A

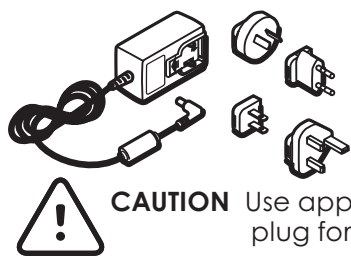


100-240V
50-60 Hz

+

107142
107143

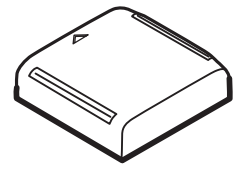





CAUTION Use appropriate power plug for your country.

1


①

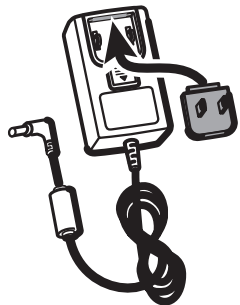


②

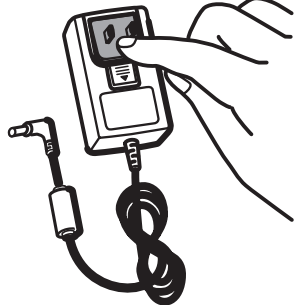


2

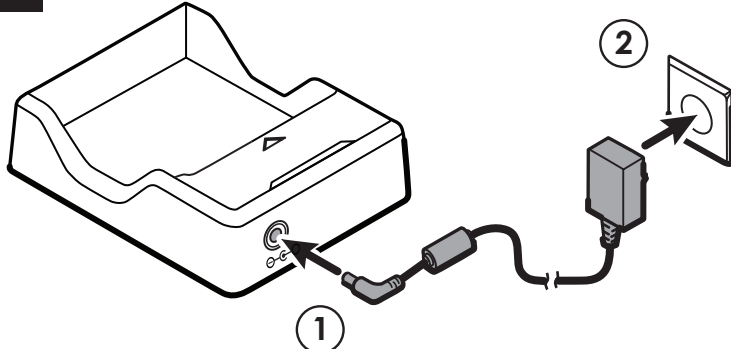



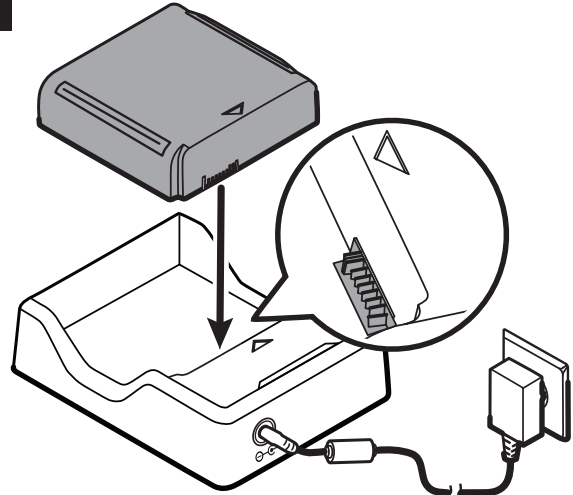

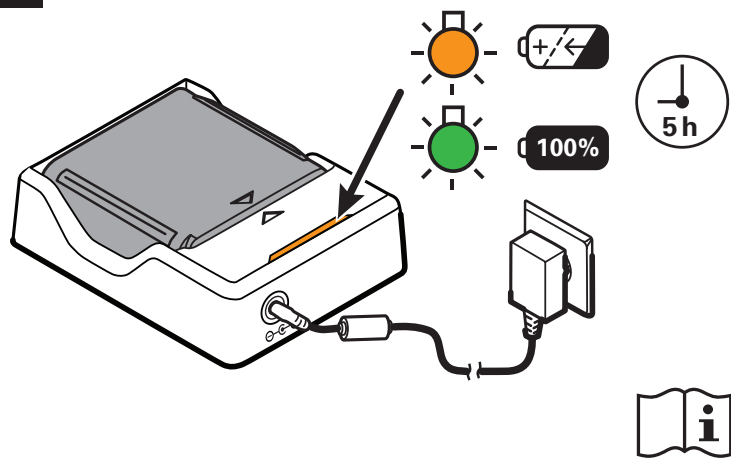

3



4



-  **WARNING** The battery charger (non-medical electrical equipment) shall be situated outside the patient environment (reference IEC 60601-1-1).
-  **WARNING** Electric shock hazard. Do not open the battery charger or attempt repairs. The battery charger has no user-serviceable internal parts. Inspection and servicing of internal parts shall only be performed by qualified service personnel.
-  **WARNING** Electric shock hazard. Before cleaning the battery charger, disconnect the AC power cord from the mains outlet and the power source.
-  **WARNING** Do not use batteries from other sources. This may result in excessive temperatures, fire, or explosion.
-  **CAUTION** Use appropriate power plug for your country.
-  **CAUTION** Do not modify this equipment without authorization of the manufacturer.