Welch Allyn® Harvey DLX Double and Triple Head Stethoscopes**Instructions for use**

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Harvey DLX

Welch Allyn Harvey DLX Double y Triple Head

Stethoscopes

Welch Allyn Harvey DLX Double e Triple Head

Stethoscopes

Welch Allyn Harvey DLX Double ja Triple Head

Stethoscopes - stethoskopit

Welch Allyn Harvey DLX Double i Triple Head

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Welch Allyn Harvey DLX Double и Triple Head

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Harvey DLX

© 2020 Welch Allyn, Inc. All rights reserved. No one is permitted to reproduce or duplicate, in any form, this instructions for use or any part thereof without permission from Welch Allyn. Welch Allyn® is a trademark of Welch Allyn, Inc. HillRom™ is a trademark of Hill-Rom Services, Inc.

Welch Allyn, Inc. ("Welch Allyn") assumes no responsibility for any injury to anyone that may result from (i) failure to properly use the product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual, or (ii) any illegal or improper use of the product.

Welch Allyn, Inc.
4341 State Street Road
Skaneateles Falls, NY 13153 USA
hillrom.com

Welch Allyn, Inc. is a subsidiary of Hill-Rom Holdings, Inc.

EC REP and EU IMPORTER

Welch Allyn Limited
Naas Business Park, Dublin Road
Naas, Co. Meath C15 AW22
Ireland

Authorized Australian Sponsor

Welch Allyn Australia Pty. Ltd.
Unit 4.01, 24-1 Lycoperdon Road
Macquarie Park, NSW 2113
Phone 1800 650 083**ENGLISH****Instructions for use****Intended use**

Welch Allyn Stethoscopes are used for diagnostic purposes by projecting the sounds associated with the heart, arteries, and veins and other internal organs.

Indications for use

Auscultation during physical assessment.

Symbol descriptionsFor information on the origin of these symbols, see the Welch Allyn symbols glossary: welchallyn.com/symbolsglossary

MD Medical device

Manufacturer

LOT Lot code

REF Reorder number

Keep dry

Fragile

WARNING The warning statements in this manual identify conditions or practices that could lead to illness, injury, or death. Warning symbols will appear with a grey background in a black and white document.

Separate collection of Electrical and Electronic Equipment. Do not dispose as unsorted municipal waste.

Warnings

WARNING Cross-contamination of stethoscope could cause patient infection. Clean and disinfect stethoscope between uses as described in the Instructions for use.

WARNING Ear tips can be swallowed and cause a choking hazard. Ensure all parts are properly attached and stored.

WARNING Stethoscope hoses can be a strangulation hazard. Keep away from children.

Residual risk

This product complies with relevant electro-magnetic interference, mechanical safety, performance, and biocompatibility standards.

However, the product cannot completely eliminate potential patient or user harm from the following:

• Harm or device damage associated with electro-magnetic hazards,

• Harm from device, function, or parameter unavailability,

• Harm from device, such as inadequate cleaning, and/or

• Harm from device exposure to biological triggers that may result in a severe systemic allergic reaction.

Chestpiece

Both the double and triple head offer a flat diaphragm and bell.

The triple head also utilises a corrugated diaphragm for additional auscultatory capabilities. Selection is accomplished by holding the chestpiece stem in one hand and rotating the chestpiece until a click is heard or felt.

Flat diaphragm

"The workhorse." Excellent for all sounds and murmurs.

Best for higher frequency heart and lung sounds. Also ideal to identify splitting of sounds, systolic clicks and ejection sounds (4.76 cm diameter).

Corrugated diaphragm

Amplifying quality. Excellent for an overview of heart sounds.

Especially good for low-frequency gallop sounds and murmurs (4.76 cm diameter).

Bell

Provides excellent detection and clarity for low and medium frequency heart sounds and murmurs such as a faint gallop or diastolic rumble (2.79 cm diameter).

Headset adjustment

Adjust the headset as follows:

1. Select ear tips that provide the maximum acoustic seal and comfort.

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