**Hillrom**

Welch Allyn®

## Elite® Stethoscope Instructions for use

Material number 772177, DIR 80025930 Ver. C  
Revision date: 2022-01

Welch Allyn, Inc.  
4341 State Street Road  
Skaneateles Falls, NY 13153

Welch Allyn, Inc. is a subsidiary of Hill-Rom Holdings, Inc.

For patent information, visit [hillrom.com/patents](https://hillrom.com/patents)

[hillrom.com](https://hillrom.com)

and EU IMPORTER  
Welch Allyn Australia Pty. Ltd.  
Navan Business Park, Dublin Road  
Navan, Co. Meath, C15 AW22  
Ireland

Authorized Australian Sponsor  
Welch Allyn Australia Pty. Ltd.  
Unit 401, 2-4 Lyonnark Park  
Macquarie Park, NSW 2113  
Phone 1800-650-083

© 2022 Welch Allyn, Inc. All rights reserved. No one is permitted to reproduce or duplicate, in any form, this manual or any part thereof without permission from Welch Allyn. Welch Allyn assumes no responsibility for any injury to anyone, or for any illegal or improper use of the product that may result from failure to use the product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual. Welch Allyn and Elite are registered trademarks of Welch Allyn, Inc.

## ENGLISH

### Intended use

Welch Allyn® Stethoscopes are used for diagnostic purposes by projecting the sounds associated with the heart, arteries, and veins and other internal organs.

### Indications for use

Auscultation during physical assessment.

### Symbol descriptions

For information on the origin of these symbols, see the Welch Allyn symbols glossary:

[welchallyn.com/symbols/glossary](https://welchallyn.com/symbols/glossary)

**WARNING** The warning statements in this manual identify conditions or practices that could lead to illness, injury, or death. Warning symbols will appear with a grey background in a black and white document.

Separate collection of Electrical and Electronic Equipment. Do not dispose as unsorted municipal waste.

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