Hillrom
 Advancing Frontline Care™
 Welch Allyn® Professional™ Stethoscope Instructions for use

Material number **REF** 772178, DIR 80025931 Ver. B Revision: 2020-07 **MD** 901039 STETHOSCOPE

Welch Allyn, Inc.
 4341 State Street Road
 Skaneateles Falls, NY 13153

Welch Allyn, Inc. is a subsidiary of Hill-Rom Holdings, Inc.
 hillrom.com

CE and EU IMPORTER
 Welch Allyn Limited
 Navan Business Park, Dublin Road,
 Navan, Co. Meath, C15 AW22
 Ireland

Authorized Australian Sponsor
 Welch Allyn Australia Pty. Ltd.
 Unit 4/1, 2-4 Lyonspark Road
 Macquarie Park, NSW 2113
 Phone 1800 650 083

© 2020 Welch Allyn, Inc. All rights reserved. No one is permitted to reproduce or duplicate, in any form, this manual or any part thereof without permission from Welch Allyn. Welch Allyn assumes no responsibility for any injury to anyone, or for any illegal or improper use of the product that may result from failure to use the product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual. Welch Allyn™ is a trademark of Welch Allyn, Inc.

ENGLISH

Intended use

Welch Allyn Stethoscopes are used for diagnostic purposes by projecting the sounds associated with the heart, arteries, and veins and other internal organs.

Indications for use

Auscultation during physical assessment

Symbol descriptions

For information on the origin of these symbols, see the Welch Allyn symbols glossary: welchallyn.com/symbolsglossary.

	WARNING The warning statements in this manual identify conditions or practices that could lead to damage to the equipment or other property, or loss of data.		Caution The caution statements in this manual identify conditions or practices that could result in damage to the equipment or other property, or loss of data.
	Not made with natural rubber latex		Separate collection of Electrical and Electronic Equipment. Do not dispose as unsorted municipal waste.
	Product identifier		Fragile
	Reorder number		Recycle
	Lot Code		Medical Device
	Global Trade Item Number		Keep dry
	Manufacturer		Consult Instructions for Use

Safety and Warnings

WARNING Cross-contamination of stethoscope could cause patient infection. Clean and disinfect stethoscope between uses as described in the Instructions for use.

WARNING Earrips can be swallowed and cause a choking hazard. Ensure all parts are properly attached and stored. **WARNING** Stethoscope hoses can be a strangulation hazard. Keep away from children.

Residual risk

This product complies with relevant electro-magnetic interference, mechanical safety, performance, and biocompatibility standards. However, the product cannot completely eliminate potential patient or user harm from the following:

- Harm or device damage associated with electro-magnetic hazards,
- Harm from mechanical hazards,
- Harm from device, function, or parameter unavailability,
- Harm from misuse errors, such as inadequate cleaning, and/or
- Harm from device exposure to biological triggers that may result in a severe systemic allergic reaction.

The Professional stethoscope offers both a diaphragm and bell which you select by holding the chestpiece stem in one hand and rotating the chestpiece with the other hand until a click is heard or felt.

Diaphragm

 "The workhorse." Excellent for all sounds and murmurs. Best for higher frequency heart and lung sounds. Also ideal for identifying split sounds, systolic clicks, and ejection sounds.


Bell

 Provides excellent detection and clarity for low and medium frequency heart sounds and murmurs (such as a faint gallop or diastolic rumble).

Headset adjustment

Adjust the headset as follows:

- Select eartips that provide the maximum acoustic seal and comfort.
- To adjust the angle of the binaurals, hold the tubing in one hand and rotate each binaural to the correct angle.
- Make sure that the binaural tubes are pointed slightly forward into the ear canal.
- To reduce spring tension, grasp each eartube at its bend and gently pull outward.
- To increase spring tension, squeeze the eartubes together.

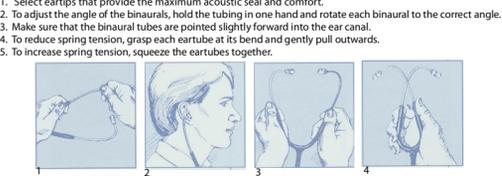Cleaning, disinfection and maintenance

Clean and disinfect the stethoscope to prevent possible cross-contamination.

Use 70% isopropyl alcohol or soapy water to wipe down the

- Eartubes and tubing
- Eartips (remove for thorough cleaning)
- Exterior of the diaphragm

- Remove the diaphragm, follow the instructions below.
- 1. Holding the diaphragm side facing you, grip the retaining rim with thumb and index finger and push the rim off the edge of the metal chestpiece.
- 2. To reassemble, align the diaphragm on the chestpiece and push the rim upward.
- 3. Continue to totally fit the rim on the chestpiece by aligning the parts in an upward motion until the rim completely snaps on.

Hold the bell and turret section in one hand, and exert pressure on the back of the diaphragm. Grip the threaded ring of the diaphragm with your other hand and turn the ring counterclockwise to loosen.

Routinely inspect the stethoscope for wear and damage. Do not use if you see signs of damage, if the instrument malfunctions, appears not to be working properly, or if you notice a change in performance. Contact Hillrom's Technical Support department for assistance.

Disposal

Users must adhere to all federal, state, regional, and/or local laws and regulations as it pertains to the safe disposal of medical devices and accessories.

If in doubt, the user of the device shall first contact Hill-Rom Technical Support for guidance on safe disposal protocols.

PATENT/PATENTS

hillrom.com/patents

May be covered by one or more patents. See above Internet address. The Hill-Rom companies are the proprietors of European, US, and other patents and pending patent applications.

Hillrom Technical Support

For information about any Welch Allyn product, contact Hill-Rom Technical Support: hillrom.com/en-us/about-us/locations.

Notice to users and/or patients in EU

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Specifications

Head diameter: 1.9in (48.3mm) Overall length: 28in (711mm) Tubing length: 17in (431.8mm)

Lot code

Lot decoder YY-JJJ where YY = last 2 digits of the year, JJJ = consecutive day of the Julian year.

Warranty

Your Professional stethoscope comes with a five-year warranty against parts and manufacturing defects. Should a material or workmanship defect be discovered, Welch Allyn will repair or replace the product at no charge. This warranty does not cover damage in shipment or failure caused by tampering, obvious carelessness or abuse.

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AVVERTENZA Cross-contamination of stethoscope could cause patient infection. Clean and disinfect stethoscope between uses as described in the Instructions for use.

AVVERTENZA Earrips can be swallowed and cause a choking hazard. Ensure all parts are properly attached and stored. **AVVERTENZA** Stethoscope hoses can be a strangulation hazard. Keep away from children.

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