Hillrom.

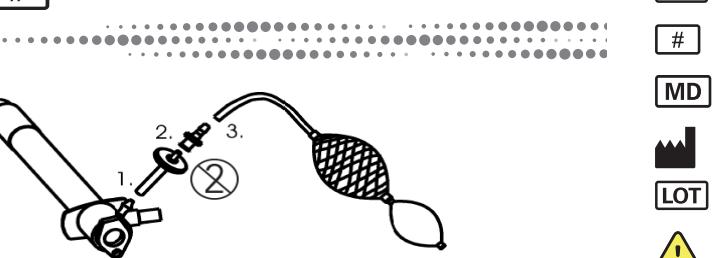
## Welch Allyn® Insufflation bulb filters

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## Welch Allyn Insufflation Bulb Filters

## Welch Allyn filtre til insufflationsbold

## Welch Allyn-insufflatieballonfilters

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© 2020 Welch Allyn, Inc. All rights reserved. No one is permitted to reproduce or duplicate, in whole or in part, this instructions for use or any part thereof without permission from Welch Allyn. Welch Allyn® is a trademark of Welch Allyn, Inc. HillRom™ is a trademark of Hill-Rom Services, Inc.

Welch Allyn, Inc. ("Welch Allyn") assumes no responsibility for any injury to anyone that may result from (i) failure to properly use the product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual, or (ii) any illegal or improper use of the product.

Welch Allyn, Inc.  
4341 State Street Road  
Skaneateles Falls, NY 13153 USA

hillrom.com

Welch Allyn, Inc. is a subsidiary of Hill-Rom Holdings, Inc.

EC/REP

CE

## and EU IMPORTER

## Welch Allyn Limited

## Navan Business Park, Dublin Road

## Navan, Co. Meath C15 AW22

## Ireland

## Authorized Australian Sponsor

## Welch Allyn Australia Pty. Ltd.

## Unit 401, 2-4 Lyonpark Road

## Macquarie Park NSW 2113

## Phone 1800 650 083

## Environmental specifications

## Temperature limitation

- Operating 10° C to 35° C (50° F to 95° F)
- Transport/Storage -20° C to 49° C (-4° F to 120° F)

## Humidity limitation

- 95% RH max

## Atmospheric pressure limitation

- 500 hPa to 1060 hPa

## Lot code

On shelf box for lot code YYYYMMDD, YYYY=Year, MM=Month, DD=Day.

On shipping package for lot code YY-JJJ, YY=Year, JJJ=Consecutive day of the Julian year.

## Notice to users and/or patients in the EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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## Lot code

## Rx ONLY

## Prescription only or "For Use by or on the order of a licensed medical professional"

## WARNING

The warning statements in this manual identify conditions or practices that could lead to illness, injury, or death. Warning symbols will appear with a grey background in a black and white document.

## Warnings

## WARNING

Intravenous Systems (IV) – Do not connect insufflation system tubing with Luer lock connectors to intravenous fluid systems or air may enter patient.

## Residual risk

This product complies with relevant electro-magnetic interference, mechanical safety, performance, and biocompatibility standards. However, the product cannot completely eliminate potential patient or user harm from the following:

- Harm or device damage associated with electro-magnetic hazards,

Harm from device, function, or parameter unavailability,

Harm from device exposure to biological triggers that may result in a severe systemic allergic reaction.

## Installation

1. Slide the free end of the silicone tubing over the sigmoidoscope barb fitting.

2. Attach the luer to the exposed filter slip connection.

3. Slide the insufflation bulb black rubber tubing over the luer.

After procedure remove the filter and its pre-attached tubing and dispose.

**NOTE** If necessary, trim the insufflation bulb tubing back 1.27 to 2.54 cm to obtain a tighter luer fit.

Replacement Filters REF 30210 (10 pack)

## Inspection

Do not use if you see signs of damage. Contact Hillrom Technical Support for assistance.

## Disposal

Users must adhere to all federal, state, regional, and/or local laws and regulations as they pertain to the safe disposal of medical devices and accessories. If in doubt, the user of the device should first contact Hillrom Technical Support for guidance on safe disposal protocols.

## PATENT/PATENTS

Welch Allyn Limited

Navan Business Park, Dublin Road

Navan, Co. Meath C15 AW22

Ireland

Authorized Australian Sponsor

Welch Allyn Australia Pty. Ltd.

Unit 401, 2-4 Lyonpark Road

Macquarie Park NSW 2113

Phone 1800 650 083

## Hillrom Technical Support

For information about any Hillrom product, contact Hillrom Technical Support at [hillrom.com/en-us/about-us/locations](http://hillrom.com/en-us/about-us/locations).

## Notice to users and/or patients in the EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## Environmental specifications

## Temperature limitation

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## Transport/Storage limitation

- -20° C à 49° C (-4° F à 120° F)

## Humidity limitation

- 95% RH max

## Atmospheric pressure limitation

- 500 hPa à 1060 hPa

## Lot code

On shelf box for lot code YYYYMMDD, YYYY=Year, MM=Month, DD=Day.

On shipping package for lot code YY-JJJ, YY=Year, JJJ=Consecutive day of the Julian year.

## Notice to users and/or patients in the EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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