Hillrom™

## 3.5V Ophthalmoscopes (Standard and Co-Axial) and Otoscopes/Throat Illuminators (Diagnostic, Pneumatic, and Operating 200, 202, 217, and 250 Series)

Rx ONLY

### Instructions for use

REF 771367 REF 80025942 Ver. B  
Revised: 2020-08

# 901081 OTHTHOALMOSCOPE, STANDARD

901079 OTSCOPIC, STANDARD

### ENGLISH

#### Intended use

The Ophthalmoscope is intended to be used to view the back of the eye, commonly called the fundus, and is a useful diagnostic aid in studying other ocular structures. It is used to support the diagnosis of ocular structure conditions and for viewing ocular abnormalities that may be indicative of conditions elsewhere in the body.

Welch Allyn Otoscopes are intended to inspect the external ear, ear canal and tympanic membrane under illumination and magnification. Secondary uses of the Otoscopes are to provide general illumination and to serve as a light source. The Pneumatic Otoscope is also used to assess flexibility of the tympanic membrane. The Operating Otoscope is designed to be used during minor otoscopic procedures for use on patients undergoing routine health assessments or minor procedures involving the ear.

These are hand-held, battery-powered devices intended to be used by trained clinicians during patient health assessments. They are for use by clinicians and medically qualified personnel for pediatric and adult patients of any gender in both ambulatory and acute care settings.

#### Description of the devices

These devices work in conjunction with 3.5V Welch Allyn power handles and associated charging and power cord system.

The Otoscope is always used with an ear tip speculum. The speculum covers the patient-contact area of the Otoscope tip and is intended to prevent patient-to-patient contamination.

#### Symbols

For information on the origin of these symbols, see the Welch Allyn symbols glossary: [welchallyn.com/symbolsglossary](http://welchallyn.com/symbolsglossary).

Temperature limitation

Humidity limitation

Atmospheric pressure limitation

Consult instructions for use

Product Identifier

Reorder number

Manufacturer

Rx ONLY

EC REP

Authorized Representative in the European Community

GTIN

Lot code

LOT

WARNING

The warning statements in this manual identify conditions or practices that could lead to illness, injury, or death. Certain symbols will appear with a yellow or grey background.

CAUTION!

The caution statements in this manual identify conditions or practices that could result in damage to the equipment or other property, or loss of data.

NOTE!

Presents clarification about an instruction or helpful information about a feature or behavior.

Medical device

MD

Separate collection of Electrical and Electronic Equipment. Do not dispose as unsorted municipal waste. bin symbol.

Single use

Keep dry

BF applied part

Warnings and cautions

WARNING

The light emitted from the ophthalmoscope is potentially hazardous.

The longer the duration of exposure, the greater the risk of ocular damage.

Exposure to light from this instrument, when operated at maximum output, can cause permanent safety guidelines after 13 minutes when it is a LED lamp (blue base) and 23 minutes when it is with a halogen lamp. Exposure times are cumulative for a 24-hour period.

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