Welch Allyn® 7114x Universal Charger

Instructions for use

REF

771364 DIR 80025958 Ver. B Revised: 2020-07

Chargeur universel 7114x

Cargador universal 7114x

Carregador Universal 7114x

7114x Universal-Ladegerät

7114x Caricatore universale

7114x Universalladdare

Universele lader 7114x

7114x Universaloplader

7114x Universal Charger

7114x Yleislaturi

7114x Universallader

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Handle and Diagnostic Head
sold separately

	Welch Allyn, Inc. 4341 State Street Road Skaneateles Falls, NY 13153 USA
	Welch Allyn, Inc. is a subsidiary of Hill-Rom Holdings, Inc. hillrom.com
	and EU IMPORTER Welch Allyn Limited Navan Business Park, Dublin Road, Navan, Co. Meath, C15 AW22 Ireland
	Authorized Australian Sponsor Welch Allyn Australia Pty. Ltd. Unit 4.01, 2-4 Lyonpark Road Macquarie Park, NSW 2113 Phone 1800 650 083

© 2020 Welch Allyn. All rights are reserved. To support the intended use of the product described in this publication, the purchaser of the product is permitted to copy this publication, for internal distribution only, from the media provided by Welch Allyn. No other use, reproduction, or distribution of this publication, or any part of it, is permitted without written permission from Welch Allyn.

Welch Allyn, Inc. ("Welch Allyn") assumes no responsibility for any injury to anyone that may result from (i) failure to properly use the product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual, or (ii) any illegal or improper use of the product. Welch Allyn® is a trademark of Welch Allyn, Inc. Hillrom™ is a trademark of Hill-Rom Services, Inc.

English

Intended use

The 7114x Universal Charger is intended to charge Welch Allyn rechargeable 2.5V and 3.5V handles.

Symbol descriptions

For information on the origin of these symbols, see the Welch Allyn symbols glossary: welchallyn.com/symbolsglossary.

 **WARNING** The warning statements in this manual identify conditions or practices that could lead to illness, injury, or death. Warning symbols will appear with a grey background in a black and white document.

	Product Identifier
	Reorder number
	Lot code
	Global Trade Item Number
	Consult instructions for use

R_x ONLY Prescription only or for use by or on the order of a licensed medical professional

	Medical device
	Manufacturer
	Authorized Representative in the European Community

	Atmospheric pressure limitation
	Humidity limitation
	Temperature limit

	AC input
	DC output
	Equipment not protected against the ingress of water.
	Separate collection of Electrical and Electronic Equipment. Do not dispose as unsorted municipal waste.

Accessories

71249	PocketScope™ Rechargeable Handle Adapter for the Universal Charger (required to charge Welch Allyn PocketScope handles in the Charger).
71310	Matching Specula Tray for convenient storage of otoscope specula, currettes or swabs. Place the Specula Tray on a flat surface or mount to a wall.
71420	Wall Bracket for mounting Universal Charger to wall - saves valuable countertop space.

Compatible devices

71500	2.5v NiCad battery handle
71670	3.5v NiCad battery handle
71900/11	3.5v Li-Ion battery handle
60713	2.5v NiCad laryngoscope handle
60835	3.5v NiCad laryngoscope handle
12800/22800	PocketScope

Warnings

WARNING Consult user's manual for additional information.

WARNING Green power light indicates that main power is connected to the appliance.

WARNING Only authorized Hillrom Service Centers can perform service.

WARNING Use of any accessories or materials not indicated in the user manual can degrade the minimum safety of the equipment.

WARNING This product complies with current required standards for electro-magnetic interference and should not present problems to other equipment nor is it affected by other devices. As a precaution, avoid using this device in close proximity to other equipment.

For information about electromagnetic compatibility (EMC), see the Welch Allyn website: welchallyn.com/emc-hndls. A printed copy of the Emissions and immunity information can be ordered from Welch Allyn for delivery within 7 calendar days.

Residual risk

This product complies with relevant electro-magnetic interference, mechanical safety, performance, and biocompatibility standards. However, the product cannot completely eliminate potential patient or user harm from the following:

- Harm or device damage associated with electro-magnetic hazards,
- Harm from mechanical hazards,
- Harm from device, function, or parameter unavailability,
- Harm from misuse error, such as inadequate cleaning, and/or
- Harm from device exposure to biological triggers that may result in a severe systemic allergic reaction.

Operation

The Universal Charger is immediately ready for use when the power cord is plugged into any grounded outlet. When illuminated, the center green power light indicates that the Charger is plugged in and ready to charge either one or two rechargeable handles.

1	Power Light	When illuminated, the center green LED indicates the Power Cord is plugged in and is supplying main power to the Universal Charger.
2	Charge Indicator Lights	When illuminated, the side yellow LEDs indicate that a handle is sensed and is charging.
3	Charging Wells	Charges the handles.
4	Power Cord	Plugs into any 100-240V, 50-60 Hz grounded outlet.

Charging handles

Place handle(s) into well(s). The yellow charge indicator LED directly under the handle well will illuminate, indicating the handle is sensed and charging. To fully charge a completely discharged battery handle, leave the handle in the charging well overnight. If you are charging your battery handle(s) for the first time, follow the charging instructions provided with your new battery handle. Once charged, placing a handle in a well after each use will top off the battery charge, maintaining the charge level of the battery. Leaving the battery handles in the charging wells overnight does not cause the handles to overcharge.

NiCad battery handles

Turn NiCad battery handles off before inserting the handle into the well.

Lithium ion handles

Once fully charged, the 71900/11 Lithium Ion handle will cause the yellow indicator LED to turn off. If desired, test the function of the charging well by placing a functioning instrument head on the handle in question, turn the handle on, and return the handle to the Universal Charger well. If the instrument lamp goes off, the handle was fully charged and the Universal Charger well is functioning properly (the yellow LED may light momentarily).

Cleaning

Periodically wipe the exterior of the Universal Charger with a damp cloth (soap and water). Use a dry cloth to clean the contacts inside the charging wells and the rechargeable battery handle bottoms.

The Universal Charger is not intended for sterilization. Do **NOT** autoclave the Universal Charger, instruments, or handles.

After cleaning, inspect the Charger assembly for visible signs of deterioration at its contacts inside the charging wells, end connector and housing. If evidence of damage or deterioration is present, discontinue use and contact Welch Allyn or your area representative.

Specifications

Model no. 7114x series

Power Cord	#18 AWG, 3 wire grounded cord, 6 feet long medical grade detachable with IEC60320 inlet.
Input	100-240V, 50-60 Hz, 100mA max, Class I, Continuous Operation
Output	2.4V - 4.0V DC 80mA max.
Weight	Approximately 660 grams (not including the rechargeable handles & cord)
Dimensions:	Charger requires only 7.25" x 3.5" desk space (not including power cord)
IPX0	Not protected against the ingress of water.
Fuses (F1, F2)	T100mA

Environmental	Temperature	Relative humidity, non-condensing	Atmospheric pressure
Storage/transport	-20° to 49°C (-4° to 120°F)	95 % R.H. Max.	500 - 1060 hPa
Operating	15° to 35°C (60° to 95°F)	75 % R.H. Max.	500 - 1060 hPa

Lot code

For lot code YY-JJJ, YY=Year, and JJJ=Consecutive day of the Julian year.

Repairs

Contact Hillrom Technical Support hillrom.com/en-us/about-us/locations for assistance which may include a Return Authorization number for product repair.

Disposal

Users must adhere to all federal, state, regional, and/or local laws and regulations as pertains to the safe disposal of medical devices and accessories. If in doubt, the user of the device shall first contact Hillrom Technical Support for guidance on safe disposal protocols.

Approvals

 Complies with IEC/UL/CSA/EN 60601-1 & 60601-1-2.

PATENT/PATENTS

hillrom.com/patents

May be covered by one or more patents. See above internet address. The Hill-Rom companies are the proprietors of European, US, and other patents and pending patent applications.

Hillrom Technical Support

For information about any Welch Allyn product, contact Hill-Rom Technical Support: hillrom.com/en-us/about-us/locations.

Warranty

Welch Allyn guarantees that this Universal Charger is free of any manufacturing defects. Welch Allyn will repair or replace, free of charge, any parts proven to be defective through causes other than misuse, neglect, damage in shipment and normal wear and tear. Welch Allyn warrants the 7114X Universal Charger will perform to original specifications for a one year period from the original date of purchase.

Welch Allyn reserves the right to make changes without notice in design, specifications and models. The only warranty Welch Allyn makes is the express written warranty extended on the sale or rental of its products.

Notice to users and/or patients in EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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